



Compatible with any  
2, 4 or 6 Point  
Spreader Bar

## PATIENT SPECIFIC SLINGS

Patient specific Double turner



Also available with  
"Key-Hole"  
Opening Clips

### GENERAL INFORMATION

Your Patient Specific (Disposable) Sling is a High Quality Product with many safety features incorporated within the material design and content of the product. The Slings are available in Standard Style (Slings without Head Support) - 2 sizes, and Fullback Style ( Slings WITH Head Support) in 5 sizes.

Each Sling is size/colour co-ordinated on the Binding, Handles, Displacement Straps.

The Slings are approved for a load of 300 Kg (660 lbs) or 500 Kg (1100 lbs) for the Extra Extra Large Size Sling. The Patient Specific Slings are compatible with most hoists (lifters) including hoists (lifters) with Key Hole Opening Clip Spreader Bars.

The Patient Specific Slings can also be used for Bathing and Showers and also in Pools.

Use with confidence - the Slings are not going to rip or tear or fall apart! Dispose of sling after bathing, etc and replace with a new sling.

The Patient Specific Slings can also be left behind under the Patient. The Slings are 100% breathable.

The Patient Specific Slings are intended to be used as a Patient Specific Product and should not be recycled and re-distributed.

**Dispose of Sling after use.**

### SLING COMPATIBILITY

The Slings with Webbing Attachments are compatible for use with any 2, 4 or 6 Point Spreader Bar Lifter System.

The Slings with Key-Hole Opening Clip Attachments (Buckles) are compatible for use with Lifter Systems where such systems provide for Key-Hole Opening Buckle-Lock Spreader Bar Attachments.

### Wash Indicator

The Patient Specific Slings incorporate a Wash Indicator Label. In the event of the sling being washed (and then recycled for use again) the Wash Indicator will have changed design so that the Care Provider is aware that the Sling has been washed.

**Do not re-use and dispose of the Sling.**

#### Prior to Washing

**WASH**

I  
N  
D  
I  
C  
A  
T  
O  
R



The Wash Indicator will high light that the Sling has been washed. Do Not Use the Sling!



#### After Washing

**WASH**

I  
N  
D  
I  
C  
A  
T  
O  
R



FOR MORE INFORMATION OR INQUIRIES PLEASE CONTACT YOUR LOCAL REPRESENTATIVE




Barrier Free Lifts  
1620 SW 17TH STREET  
OCALA FL34471  
1-800-582-8732  
USA

# Double Turner Instructions


1.

1. Fold the strap with the velcro hook & loop under the narrow material on one end. With palm down slide under the lower back of the patient by compressing the mattress. You will hold the material and velcro under your palm by simply curling your thumb under the edge.

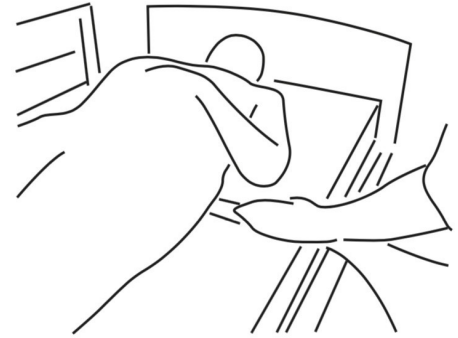


2.

2. Slide the strap to the other side of. Open the velcro loop and attach it to the bed frame. \*If you do not have access to the frame you "may" be able to use a side rail depending on the model.

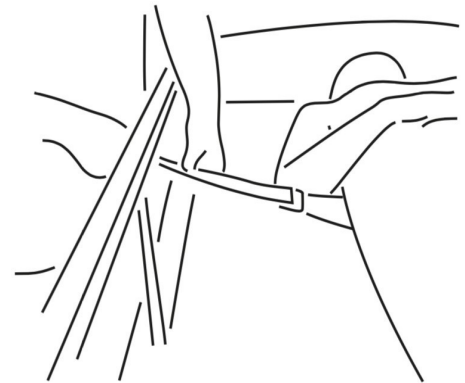

3. Attach the Double Turner sling straps to the lift hooks. Start lifting the sling.

3.

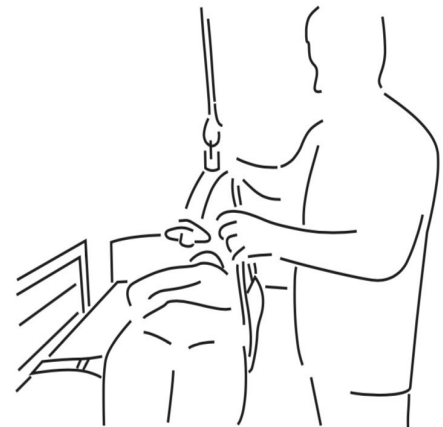

4. The lift hooks should be parallel to the patient's back side. Cross or straighten legs before turning. Support the patient while turning, keeping spinal in alignment by holding the shoulder as the sling raises.

4.

5. After completing turning of the patient, lower the sling. Remove the velcro loop and hook from the bed frame and fasten it onto itself before you pull it out from underneath the patient. or lower the sling and when ready to turn to the opposite side reverse the process.

